QUICK START GUIDE

One mounting arm

WARRANTY

AbleNet manufactured products include a 2-year limited warranty. This warranty is against defects in materials and manufacturing for 2-years from the date of purchase. Full warranty details available at www.ablenetinc.com.

AbleNet, Inc.
2625 Patton Road
Roseville, MN 55113
United States of America
(651) 294-3101
ablecare@ablenetinc.com
www.ablenetinc.com

Made with recycled content

EN March 11, 2025 5:04 PM

Product Registration

Registering your product provides you access to ableCARE, product updates, and resources for your product. Scan the QR code below to register your product.

Getting Started

Scan the QR code below to watch a short getting started video or follow the instructions listed.

To get started:

1. Attach mounting arm to table, wheelchair, or other surfaces. Turning the handle on the clamp counterclockwise will loosen or open the clamp to fit on larger surfaces. Turning the handle on the clamp clockwise will tighten the clamp.
2. Attach threaded bolt side of mounting plate to end of mounting arm.
3. Use friction knob handle to adjust the mounting arm. Turning the friction knob handle counterclockwise to loosen and then clockwise to tighten after positioning the mounting arm.

To request a different mounting arm, clamp, or mounting plate, contact ableCARE at ablecare@ablenetinc.com or (651) 294-3101.

This device can do more! Full Instructions for Use are available at www.ablenetinc.com.

ableCARE™

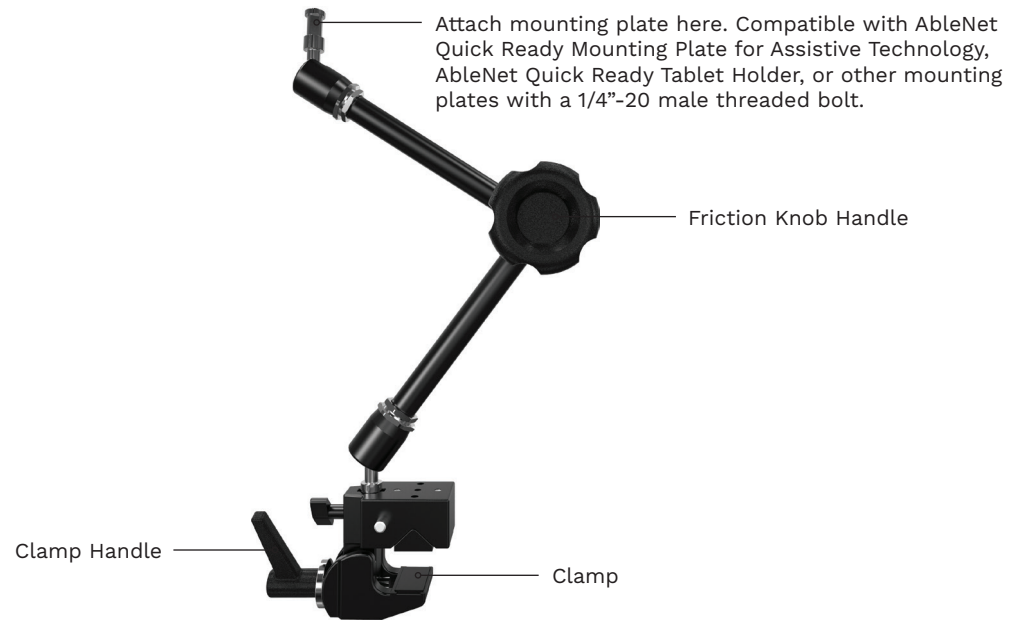
Download the ableCARE app to access live support from the ableCARE Product Success team, an online knowledge base filled with videos and getting started information, and other resources.

Scan the QR code below to download the free ableCARE app to your phone or tablet.

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